

MotionTrak™ System Directions & Test Result Printing

Launch MotionTrak Program

Select LOG IN

Select MAXIMUM

Enter Password, Select OK

Select TEST

Select NEW PATIENT (Red fields must be completed) or LOG for existing patients in database

Select WSRTest, confirm patient information, Select YES

- Perform Conditions (1 – 6)
- Select YES when prompted with Do you want to save the data? after completing each condition

*If at any time the patient begins to sway too much, use the stop button to stop the test. Select **SAVE THE DATA**. **Do not allow the patient to fall**. On the analysis, a red bar will be shown in the Equilibrium Score that indicates that the patient would not have been able to complete the test procedure.

After Condition 6, Select CLOSE

Select ANALYZE

1. Select the date of test
2. Highlight one test (eg. Romberg Open Stance Eyes Closed)
3. Check EQUILIBRIUM SCORES
4. Check DISPLAY

Select PRINT ICON